

## Sending Digital Files to



P: 877 282-3462

A: 347 Wyoming Ave Wyoming, PA 19644

If work or scans should be forwarded to another Shaw Lab location, please indicate it on your Rx. Offices can always email exported and zipped .stl or DICOM files to

[scanning-pennsylvania@shawlabgroup.com](mailto:scanning-pennsylvania@shawlabgroup.com)

## Medit Link

- Request a partnership with "Shaw Labs Pennsylvania" using the email [scanning-pennsylvania@shawlabgroup.com](mailto:scanning-pennsylvania@shawlabgroup.com)

## iTero®

aka mycadent, iRecord,  
iTero Restorative

### Option 1

- Select "Find a laboratory" on your iTero Scanner
- Connect with Best Value Ceramics using our **Company ID 90452**
- After connecting to us as a Lab, select "**Shaw Laboratories Corp**" on your scanner when sending files

### Option 2

- Call 1 800 577 8767
- Select Option 1
- Request that "Shaw Dental Laboratories Corp." is added to your scanner
- Identify our lab using our phone number **570 693-2252**
- After **Shaw Laboratories Corp.** has been added, restart your scanner
- After connecting to us as a lab, select "**Shaw Group**" on your scanner when sending files

## TRIOS®

aka 3Shape

- Go to [us.3shapecommunicate.com](http://us.3shapecommunicate.com) in a web browser
- Connect with Best Value Ceramic as your Lab by searching [scanning-pennsylvania@shawlabgroup.com](mailto:scanning-pennsylvania@shawlabgroup.com)
- After connecting to us as a lab, select " **Shaw Labs Pennsylvania** " when sending files



previously known as 3M True  
Definition, 3M ESPE

- Contact Midmark Technical Support at 1-844-856-1231 and request "**Shaw Laboratories (PA)**" to be added to your account